# COAGULATION CONTROL I, II

## CONTROL DE COAGULACIÓN I, II

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### PRODUCT DESCRIPTION

The Coagulation Control is a lyophilized human plasma with stabilizer suitable for the quality control of the clinical laboratories. The product is intended for intralaboratory quality control purposes only and is supplied with intervals of suggested acceptable values.

### CONTENTS AND COMPOSITION

**Coagulation Control.** Lyophilized human plasma (for 1 mL).

*Components from human origin have been tested and found to be negative for the presence of antibodies anti-HIV and anti-HCV, as well as for HBs antigen. However, they should be handled cautiously as potentially infectious.*

### INSTRUCTIONS FOR USE

1. Open a vial very carefully, avoiding any loss of the lyophilized material.
2. Pipette exactly 1.00 mL of distilled water into the vial. The component values depend on the accuracy of this reconstitution step.
3. Close the vial with the stopper and let stand for 30 minutes at room temperature.
4. Swirl gently, avoiding the formation of foam, to ensure complete dissolution of contents.
5. The reconstituted control plasma is to be treated like the patient plasma.

### STORAGE AND STABILITY

Store at 2-8°C.

The lyophilized Coagulation Control is stable until the expiration date given in the label.

The components of the reconstituted Coagulation Control are stable for 4 hours at 22°C.

Reconstituted Coagulation Control is stable for at least 8 days at -20°C. Freeze only once.

### SPECIFIED VALUES

The assigned concentration values for components and their traceability are shown in the enclosed tables. Traceability can be assured only if the BioSystems reagents and recommended measurement procedures are used.

The intervals of suggested acceptable values have been calculated from previous experience in interlaboratory variability and are given for orientation only; each laboratory should establish its own precision parameters.

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